



Description

3X Neutral Plus is a highly concentrated, non-corrosive, neutral pH detergent utilizing a specially designed enzyme blend paired with a next generation water conditioning package to break down organic soils and safely clean instruments in a variety of water conditions. It is designed for use in washer- disinfectors, cart washers, and ultrasonics.

Features

- Concentrated for economy
- High performance cleaning in a neutral pH range
- Excellent in varying water conditions
- Compatible with most Sterile Processing Department substrates

Directions for Use

Washer-Disinfectors and Cart Washer Applications: Dose 3X Neutral Plus between 0.05-1.0 oz./gal. of water (0.4-7.9 mL/L), according to water quality and typical soil load. Consult the machine manufacturer's directions for use of the specific machine being used.

Ultrasonic Applications: Dose 3X Neutral Plus between 0.20-1.0 oz./gal. of water (1.6-7.9 mL/L), according to water quality and typical soil load. Consult the ultrasonic manufacturer's directions for use of the specific model being used.

Sink (Manual) Applications: Dose 3X Neutral Plus between 0.20-2.0 oz./gal. of water (1.6-15.8 mL/L). Soak duration should be determined by the individual facility's standard operating procedures. Intercon recommends a minimum soak time of 2 minutes.

Endoscopes: Dose 3X Neutral Plus between 0.20-1.0 oz./gal. of water (1.6-7.9 mL/L). Process the soiled scopes according to the manufacturer's directions for use or to the specifications determined by the individual facility's standard operation procedure.

Safety

Be sure to don appropriate Personal Protection Equipment.

This product is intended for industrial and institutional use only. Not intended for consumer use. Consult the Safety Data Sheet (SDS) for more detailed information pertaining to safety, handling, storage, and waste disposal methods.

Do not add bleach or any other chemical to 3X Neutral Plus. Combining chemicals outside of the directions for use could result in loss of efficacy or adverse chemical reactions.

The product is freeze/thaw stable. If frozen, thaw to room temperature and shake or stir.

Once the container is empty, rinse and dispose of properly. Do not reuse this container. Any detergent waste should be disposed of according to Federal, State, and/or local regulations.



3X Neutral Plus

Concentrated Neutral pH Detergent
for Sterile Processing

Properties

Color: Clear, colorless to amber liquid

Odor: Characteristic odor

Product pH: 6.0-8.0

Store below 100°F (38°C)

Use up to 23 months from date of manufacture

Package Sizes Available

4x1 Gallon	FICCB-CP-04X1-8703
2.5 Gallon	FICCB-CP-2.5G-8703
5 Gallon	FICCB-CP-005T-8703
15 Gallon	FICCB-CP-0015-8703



Refer to Product Label and SDS for Additional Information.

314-771-6600 • 800-325-9218 • interconchemical.com

1100 Central Industrial Dr, St. Louis, MO 63110

2025 Intercon Chemical Company.

