



## Description

**One & Done!** is a high performance, free rinsing detergent designed to break down tough surgical soils, handle difficult water conditions and complete the job of both an enzyme detergent and a traditional detergent in a single wash phase. This non-corrosive formulation is to be used in washer-disinfectors and cart washers.

## Features

- Single-phase detergent
- Saves on energy / water usage
- Saves on cycle time
- Concentrated for economy
- Compatible with most Sterile Processing Department substrates

## Directions for Use

For washer-disinfectors and cart washer applications, dose One & Done! between 0.1-1.0 oz./gal. of water (0.8-8.0 mL/L), according to water quality and typical soil load. Adjust phase settings to eliminate the enzyme phase on the wash cycle. Set detergent phase to run for 4 minutes at 120°-130°F (49° - 54°C). Detergent can be dosed at machine's ambient temperature.

Consult the machine manufacturer's directions for use of the specific machine being used.

## Safety

Be sure to don appropriate Personal Protection Equipment.

This product is intended for industrial and institutional use only. Not intended for consumer use. Consult the Safety Data Sheet (SDS) for more detailed information pertaining to safety, handling, storage, and waste disposal methods.

Do not add bleach or any other chemical to One & Done!. Combining chemicals outside of the directions for use could result in loss of efficacy or adverse chemical reactions.

The product is freeze/thaw stable. If frozen, thaw to room temperature and shake or stir.

Once the container is empty, rinse and dispose of properly. Do not reuse this container. Any detergent waste should be disposed of according to Federal, State, and/or local regulations.



# One & Done!

High Performance, Single-Step Detergent  
for Sterile Processing

## Properties

**Color:** Clear, light amber liquid

**Odor:** Characteristic odor

**Product pH:** 6.5-8.5

Store below 100°F (38°C)

Use up to 23 months from date  
of manufacture

## Package Sizes Available

4x1 Gallon	FICCB-CP-04X1-0798
5 Gallon	FICCB-CP-005T-0798
15 Gallon	FICCB-CP-0015-0798



Refer to Product Label and SDS for Additional Information.

314-771-6600 • 800-325-9218 • interconchemical.com

1100 Central Industrial Dr, St. Louis, MO 63110

2025 Intercon Chemical Company.

