

Cleaner

Description

Clear Choice! is a high performance, multi-enzyme detergent, designed to break down the tough surgical soils that matter (proteins, fats, oils and greases). Its unique formulation is free from dyes and fragrance and is designed to remain clear in soiled sink solutions, while also providing high powered cleaning in a wide range of temperatures. Clear Choice! is truly multi-functional as it can be used manually and in sinks, ultrasonics, washer-disinfectors, and cart washers.

Features

- Multi-functional enzymatic detergent
- Highly effective in a wide temperature range 65°-140°F (18°-60°C)
- Clear in soiled sink solutions - low clouding
- Novel protease and lipase blend
- Free from fragrances and dyes
- Concentrated for economy
- Compatible with most Sterile Processing Department substrates

Directions for Use

Washer-Disinfectors and Cart Washer Applications: Dose between 0.05-1.0 oz./gal. of water (0.4-7.9 mL/L), according to water quality and typical soil load. Consult the machine manufacturer's directions for use of the specific machine being used.

Ultrasonic Applications: Dose between 0.20-1.0 oz./gal. of water (1.6-7.9 mL/L), according to water quality and typical soil load. Consult the ultrasonic manufacturer's directions for use of the specific model being used.

Sink (Manual) Applications: Dose between 0.20-2.0 oz./gal. of water (1.6-15.8 mL/L). Soak duration should be determined by the individual facility's standard operating procedures. Intercon recommends a minimum soak time of 2 minutes.

Endoscopes: Dose between 0.20-1.0 oz./gal. of water (1.6-7.9 mL/L). Process the soiled scopes according to the manufacturer's directions for use or to the specifications determined by the individual facility's standard operating procedures.

Safety

Be sure to don appropriate Personal Protection Equipment.

This product is intended for industrial and institutional use only. Not intended for consumer use. Consult the Safety Data Sheet (SDS) for more detailed information pertaining to safety, handling, storage, and waste disposal methods.

Do not add bleach or any other chemical to Clear Choice!. Combining chemicals outside of the directions for use could result in loss of efficacy or adverse chemical reactions.

The product is freeze/thaw stable. If frozen, thaw to room temperature and shake or stir.

Once the container is empty, rinse and dispose of properly. Do not reuse this container. Any detergent waste should be disposed of according to Federal, State, and/or local regulations.



Clear Choice!

High Performance Enzymatic Detergent
for Sterile Processing

Properties

Color: Clear, light yellow to amber liquid

Odor: Characteristic odor

Product pH: between 6.0-7.5

Store below 100°F (38°C)

Use up to 23 months from date of manufacture

Package Sizes Available

4x1 Gallon	FICCB-CP-04X1-8802
2.5 Gallon	FICCB-CP-2.5G-8802
5 Gallon	FICCB-CP-005T-8802
15 Gallon	FICCB-CP-0015-8802



Refer to Product Label and SDS for Additional Information.

314-771-6600 • 800-325-9218 • interconchemical.com

1100 Central Industrial Dr, St. Louis, MO 63110

2025 Intercon Chemical Company.

