

# Cleaner

## Description

**Instra-Lube Plus** is a concentrated, steam and ethylene oxide permeable, instrument lubricant specifically designed for immersible applications to prolong instrument lifespan. Instra-Lube Plus is primarily designed for use in the thermal phase of washer-disinfectors and cart washers, and manual applications when appropriate.

## Features

- Prevents discoloration, corrosion, and rusting
- Preventative layer does not interfere with sterilization, steam and ethylene oxide permeable
- Increases instrument lifespan
- Ensures joints and hinges function properly
- Does not need to be rinsed or removed prior to sterilization

## Directions for Use

For washer-disinfector and cart washer applications, dose Instra-Lube Plus between 0.05-0.33 oz./gal. of water (0.4-2.5 mL/L). Instra-Lube Plus should always be diluted using reverse osmosis or deionized water and should be dosed above 150°F (65°C). Consult the machine manufacturer's directions for use of the specific machine being used.

For manual applications, dilute product 1:6 with deionized or reverse osmosis water. Soak instruments in solution for a minimum of 1 minute. Rinsing or wiping the instrument is not necessary. Instruments are now ready for sterilization. Discard solution daily.

## Safety

Be sure to don appropriate Personal Protection Equipment.

This product is intended for industrial and institutional use only. Not intended for consumer use. Consult the Safety Data Sheet (SDS) for more detailed information pertaining to safety, handling, storage, and waste disposal methods.

Do not add bleach or any other chemical to Instra-Lube Plus. Combining chemicals outside of the directions for use could result in loss of efficacy or adverse chemical reactions.

**DO NOT FREEZE. This product is NOT freeze/thaw stable. If product is or has been frozen, DO NOT USE. Discard in appropriate containers.**

Once the container is empty, rinse and dispose of properly. Do not reuse this container. Any detergent waste should be disposed of according to Federal, State, and/or local regulations.



# Instra-Lube Plus

Concentrated Instrument Lubricant  
for Sterile Processing

## Properties

**Color:** Milky, white liquid

**Odor:** Mild fragrance

**Product pH:** between 7.0-8.0

Store below 120°F (49°C)

**DO NOT FREEZE**

Use up to 23 months from date  
of manufacturer

## Package Sizes Available

4x1 Gallon	FICCB-CP-04X1-8501
2.5 Gallon	FICCB-CP-2.5G-8501
5 Gallon	FICCB-CP-005T-8501
15 Gallon	FICCB-CP-0015-8501



Refer to Product Label and SDS for Additional Information.

314-771-6600 • 800-325-9218 • [interconchemical.com](http://interconchemical.com)

1100 Central Industrial Dr, St. Louis, MO 63110

2025 Intercon Chemical Company.

