

MAD Cleaner

Description

Opti Neutral is a neutral pH, enzyme-free detergent specifically formulated to clean ophthalmic instruments. This product is non-foaming, free rinsing, and suitable for use in mechanical washers including washer-disinfectors, cart washers, and ultrasonic cleaners as well as in manual applications. It is compatible with a variety of substrates typically used in sterile processing, including stainless steel, aluminum, various plastics, and glassware. Opti Neutral includes a water-conditioning package designed to operate effectively in various local water conditions.

Features

- High-performance cleaning without enzyme or alkalinity
- Non-corrosive to aluminum, stainless steel, plastics, glassware, and most other Sterile Processing Department substrates
- Excellent in varying water conditions
- Free rinsing

Directions for Use

Washer-Disinfectors and Cart Washer Applications: Dose Opti Neutral between 0.2-2.0 oz./gal. of water (2.0-16 mL/L), according to water quality and typical soil load. Consult the machine manufacturer's directions.

Ultrasonic Applications: Dose Opti Neutral between 1.0-2.0 oz./gal. of water (8.0-16 mL/L), according to water quality and typical soil load. Consult the ultrasonic manufacturer's directions for use of the specific model being used.

Manual Applications: Dose Opti Neutral 1.0-2.0 oz./gal. of water (8.0-16 mL/L) into the sink using a dosing system or 1 oz. pump. Allow instruments to soak or use preferred cleaning apparatus. Replace solution once observably soiled.

Safety

Be sure to don appropriate Personal Protection Equipment.

This product is intended for industrial and institutional use only. Consult the Safety Data Sheet (SDS) for more detailed information pertaining to safety, handling, storage, and waste disposal methods.

Do not add bleach or any other chemical to Opti Neutral. Combining chemicals outside of the directions for use could result in loss of efficacy or adverse chemical reactions.

The product is freeze/thaw stable. If frozen, thaw to room temperature and shake or stir.

Once the container is empty, rinse and dispose of properly. Do not reuse this container. Any detergent waste should be disposed of according to Federal, State, and/or local regulations.



Opti Neutral

Non-Enzymatic, Neutral pH Detergent
for Ophthalmic Sterile Processing

Properties

Color: Clear, colorless to light yellow liquid

Odor: Mild odor

Product pH: 6.0-8.0

Store below 120°F (49°C)

Use up to 23 months from date of manufacture

Package Sizes Available

4x1 Gallon	FICCB-CP-04X1-8704
2.5 Gallon	FICCB-CP-2.5G-8704
5 Gallon	FICCB-CP-005T-8704
15 Gallon	FICCB-CP-0015-8704



Refer to Product Label and SDS for Additional Information.

314-771-6600 • 800-325-9218 • interconchemical.com

1100 Central Industrial Dr, St. Louis, MO 63110

2025 Intercon Chemical Company.

