



Description

3X Enzyme Plus is a highly concentrated enzymatic detergent, with a three-enzyme blend to specifically target protein, fat, oil, and starch based soils. Its combination of surfactants, water conditioners, builders, and stabilizers combine with the enzyme blend to make 3X Enzyme Plus balance for various water environments and soil loads. It is suitable for use in sinks, ultrasonics, washer-disinfectors, and cart washers.

Features

- Concentrated for economy
- Multi-enzyme blend targets a variety of soils
- Excellent in varying water conditions
- Compatible with most Sterile Processing Department substrates

Directions for Use

Washer-Disinfectors and Cart Washer Applications: Dose 3X Enzyme Plus between 0.05-1.0 oz./gal. of water (0.4-7.9 mL/L), according to water quality and typical soil load. Consult the machine manufacturer's directions for use of the specific machine being used.

Ultrasonic Applications: Dose 3X Enzyme Plus between 0.20-1.0 oz./gal. of water (1.6-7.9 mL/L), according to water quality and typical soil load. Consult the ultrasonic manufacturer's directions for use of the specific model being used.

Sink (Manual) Applications: Dose 3X Enzyme Plus between 0.20-2.0 oz./gal. of water (1.6-15.8 mL/L). Soak duration should be determined by the individual facility's standard operating procedures. Intercon recommends a minimum soak time of 2 minutes.

Endoscopes: Dose 3X Enzyme Plus between 0.20-1.0 oz./gal. of water (1.6-7.9 mL/L). Process the soiled scopes according to the manufacturer's directions for use or to the specifications determined by the individual facility's standard operating procedures.

Safety

Be sure to don appropriate Personal Protection Equipment.

This product is intended for industrial and institutional use only. Not intended for consumer use. Consult the Safety Data Sheet (SDS) for more detailed information pertaining to safety, handling, storage, and waste disposal methods.

Do not add bleach or any other chemical to 3X Enzyme Plus. Combining chemicals outside of the directions for use could result in loss of efficacy or adverse chemical reactions.

The product is freeze/thaw stable. If frozen, thaw to room temperature and shake or stir.

Once the container is empty, rinse and dispose of properly. Do not reuse this container. Any detergent waste should be disposed of according to Federal, State, and/or local regulations.



3X Enzyme Plus

Concentrated Enzymatic Detergent
for Sterile Processing

Properties

Color: Clear, blue liquid

Odor: Characteristic odor

Product pH: between 7.0-8.5

Store below 100°F (38°C)

Use up to 23 months from date
of manufacture

Package Sizes Available

4x1 Gallon	FICCB-CP-04X1-8803
2.5 Gallon	FICCB-CP-2.5G-8803
5 Gallon	FICCB-CP-005T-8803
15 Gallon	FICCB-CP-0015-8803



Refer to Product Label and SDS for Additional Information.

314-771-6600 • 800-325-9218 • interconchemical.com

1100 Central Industrial Dr, St. Louis, MO 63110

2025 Intercon Chemical Company.

